Bensektion for Puslebord Medi 2

Brugermanual

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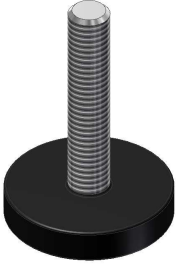
User Manual

Mounting Instructions

Keep this manual with the product at all times

Contents

General information	15
Mounting instructions	17
Maintenance	22
Spare parts	23
Complaints.....	24

General information

Product information

It is important to read this manual prior to mounting and use.

Product : Leg section for Nursing Table Medi 2

Manufacturer: Ropox, DK-4700 Næstved, Tel. +45 55 75 05 00

Application – The leg section is used when the wall is not strong enough to carry the wall-mounted nursing table.

Item number

Length of nursing table	Item number
90 cm and 130 cm	50-50687

Nursing table Medi 2, 90 cm

Nursing table Medi 2, 130 cm

General information

Technical data

Material – frame: Welded steel pipes and steel sheet St.37,
Surface treatment: Powder coated, white RAL 9010 mat, chromate

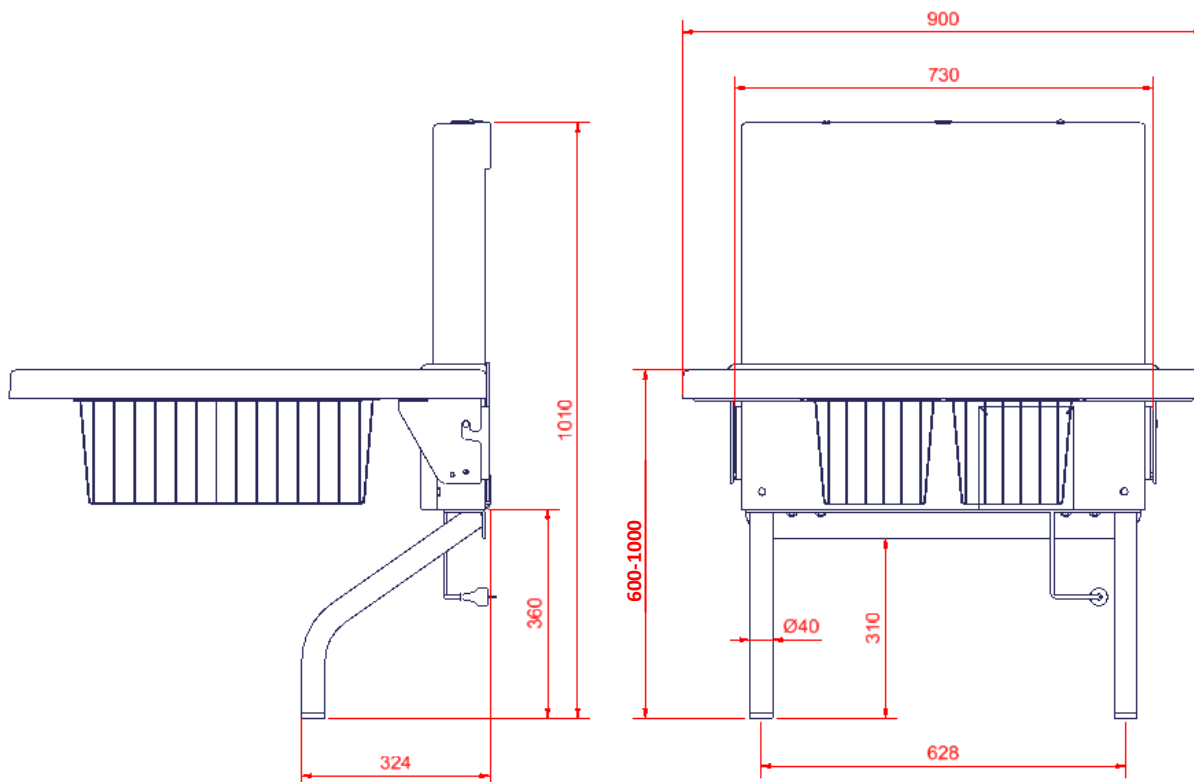
Before using the product

Before using this product, make sure that the mounting instructions have been followed carefully and that all bolts and nuts have been tightened.


Mounting instructions

Important information prior to mounting

Dimension sketch of nursing tables with leg section.



Nursing table Medi 2, 90 cm

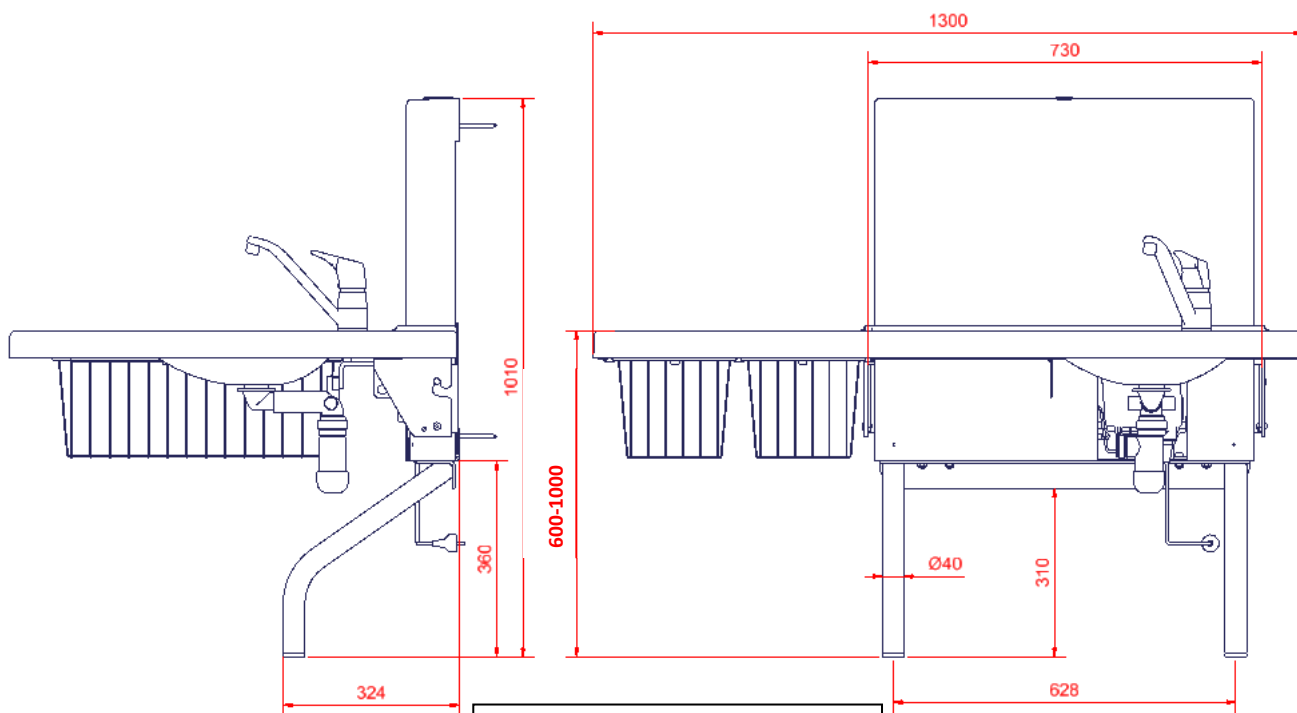Nursing table Medi 2, 130 cm

Mounting instructions



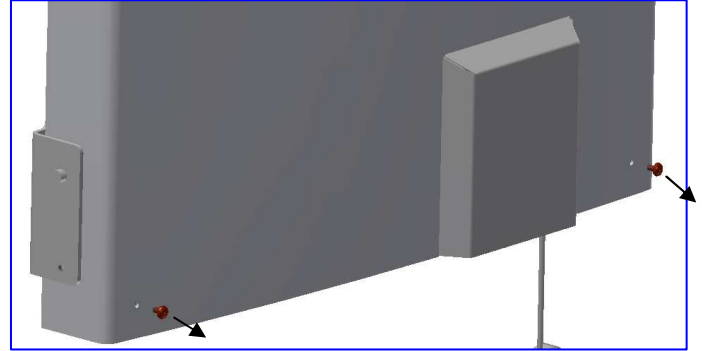
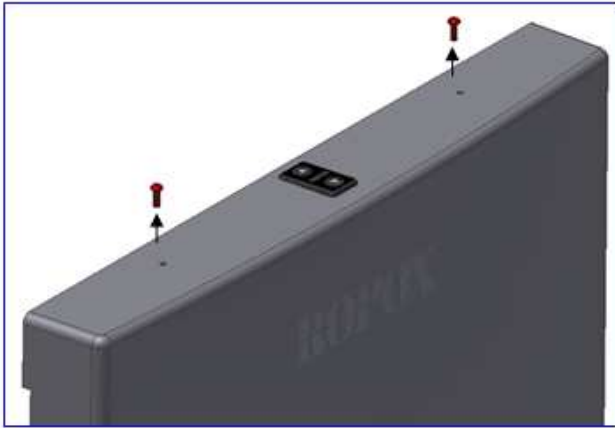
1. Dismounting of cover

Loosen the four bolts of the cover and remove the cover

Mounting instructions

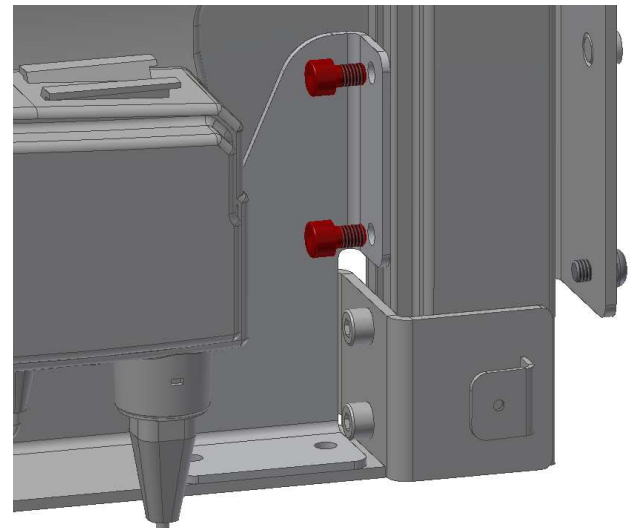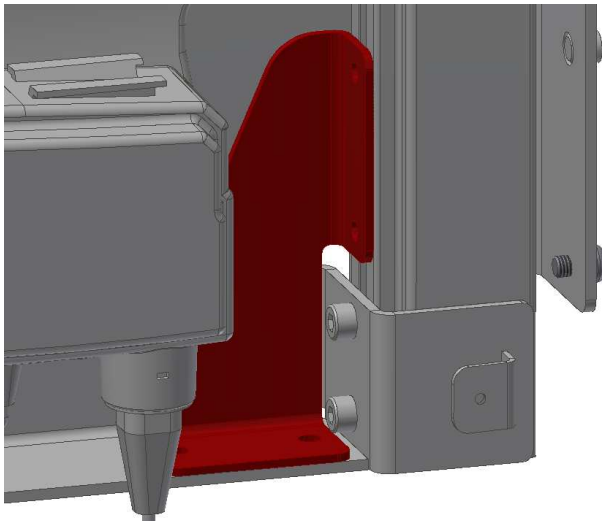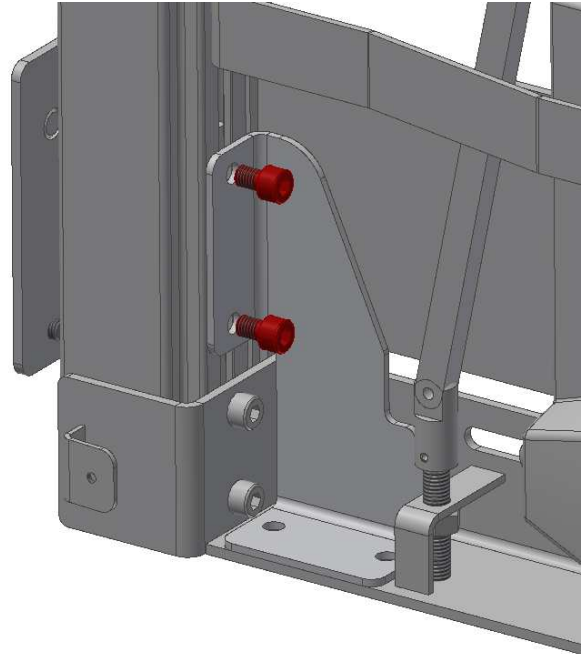
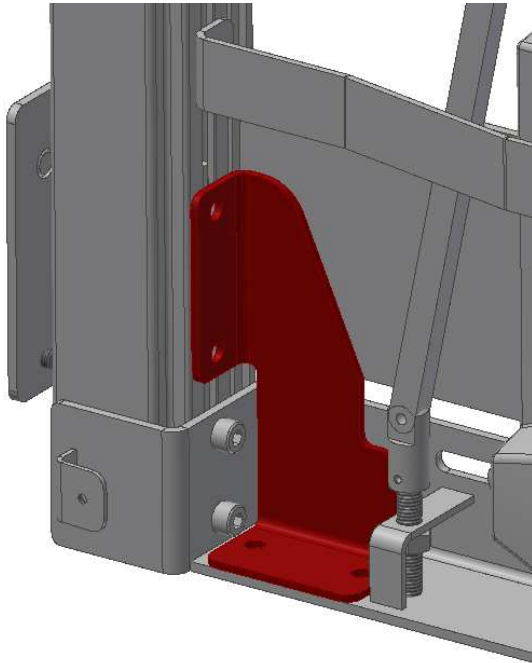
2. Mounting of reinforcement fittings

Place and fasten the two reinforcement fittings on the aluminium profiles using 4 x M8x15


Mounting instructions

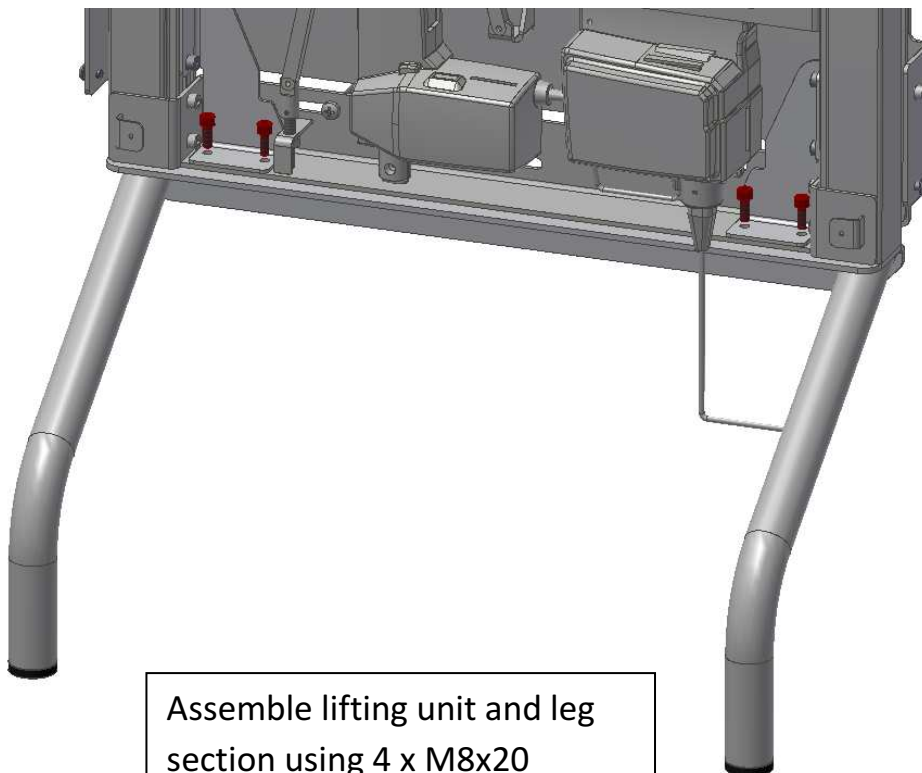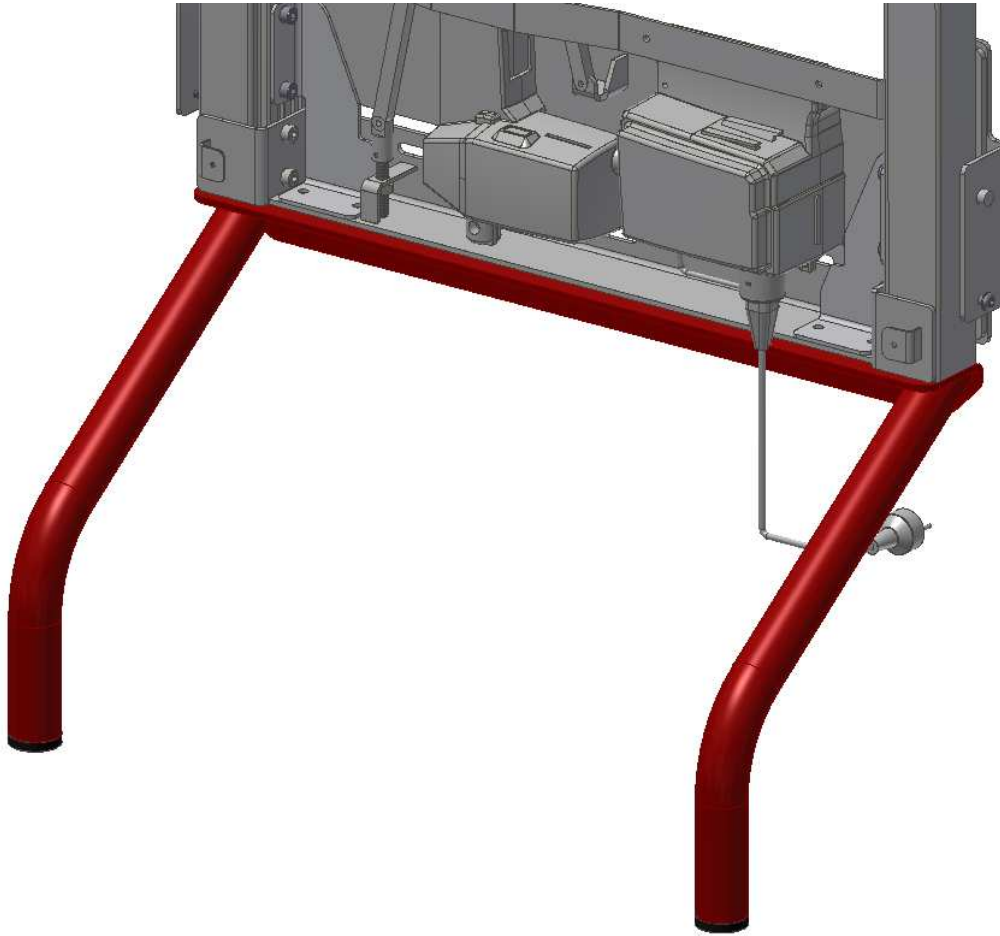3. Mounting of legs

Assemble lifting unit and leg section using 4 x M8x20

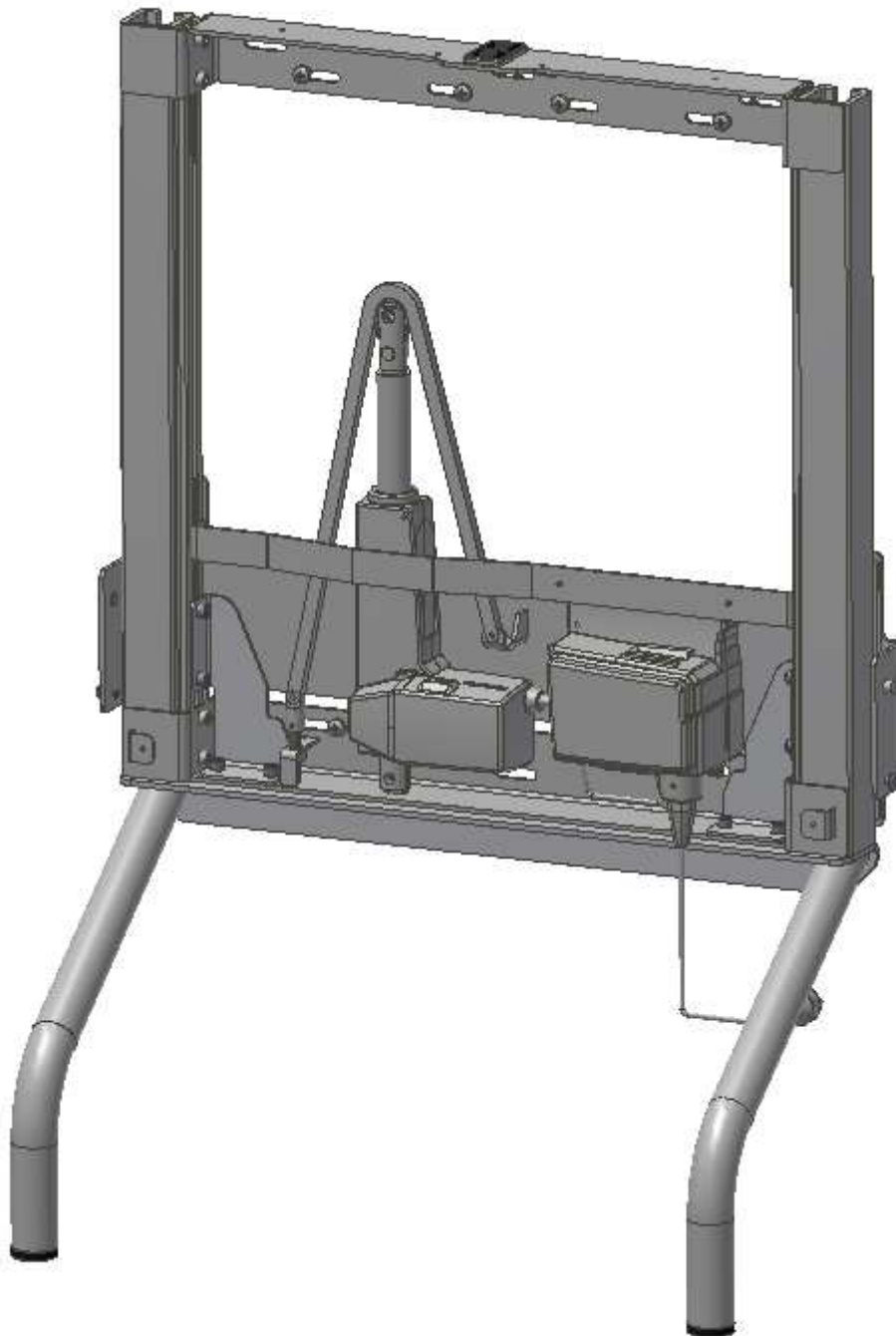Mounting instructions

The leg section has been fitted and the lifting unit may be fastened on the wall according to the mounting instructions of the nursing table

- i** Ropox recommends testing of the functions of the nursing table and retightening of all bolts and nuts once a year.

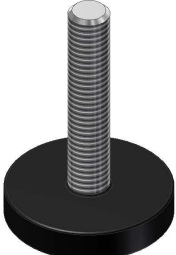
In case of defects, please contact Ropox as soon as possible, and do not use the nursing table until the problem has been solved/remedied.

!! Maintenance of the nursing table should only be carried out by competent personnel !!

If you do not want to carry out the performance test yourself, you may contact Ropox to arrange a visit by a service technician. See contact information at the end of these instructions.

Spare parts

Spare parts

96100827	Adjusting screw M8x30 (Ø44)	
----------	-----------------------------	---

Complaints

Please see General Terms of Sale and Delivery on www.ropox.com

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Ropox A/S